



Summary of Instructions and Warnings for Using the **SPLASH! – Aquatic Patient Hoist**

Read and Follow all User Instructions listed in the Owner's Manual before use
Read and Follow all Caution, Warning labels located on the hoist before use

Warnings and Safety Summary



DANGER – FAILURE TO FOLLOW THESE WARNINGS, INSTRUCTIONS AND THE OWNER'S MANUAL MAY RESULT IN SERIOUS INJURY OR DEATH

WARNING

- **ADULT USE ONLY** – Use by children with adult supervision only. Only persons healthy enough for water activities should use the SPLASH!. Users should consult with their physician to determine if water activities are appropriate for the User.
- Seat belt is to be used at all times during user transfer.
- **NO JUMPING/DIVING/CLIMBING** – Do not jump/dive from or climb on hoist.
- Limit for use is **ONE PERSON** weighing less than the maximum weight limit of – 400lbs (181kg) for the SPLASH! And SPLASH! Hi/Lo and 300lbs (136kg) for the SPLASH! 300 and SPLASH! Extended Reach. Use only for transfer of user not as a hoist or crane.
- **PINCH/CRUSH HAZARD** – Always keep hands and fingers clear of hoist arms during use.
- The battery must be completely removed from hoist and located away from pool deck for charging.

CAUTION

- **BATTERY CHARGE LEVEL** – Check Battery Level before each use. Battery must show **GREEN** >50% charge level before use. When **YELLOW** <50% charge level is shown the battery must be fully charged before use. When **RED** <25% charge level is shown the hoist must not be used and the battery must be fully charged before use.
- **DAMAGE MAY RESULT** – Do not use the seat or seat frame to move or position the Splash!.
- **DAMAGE MAY RESULT** – Do not allow excessive moisture to collect in the battery case, it can affect battery and hoist performance and could lead to battery failure and/or the hoist failing to operate.



S.R. Smith, LLC
PO Box 400
1017 SW Berg Parkway
Canby, Oregon 97013 USA
Phone: 503-266-2231 Fax:
503-266-4334
www.srsmith.com
Assembled in the USA




QNET B.V.
Hommerterweg 286
6436 AM Amstenrade
The Netherlands
qnet@ce-mark.com
qnet@ce-authorizedrepresentative.eu



TRANSFERRING

- Do transfer onto seat, ensuring user's weight is centered on seat.
- Do fasten seat belt or optional stability vest.
- Do rotate seat to 12:00 position, over water.
- Do lower seat into pool.
- Do unfasten seat belt or optional stability vest – transfer into water.
- Do when finished, return to seat, ensuring user's weight is centered on seat.
- Do fasten seat belt or optional stability vest.
- Do rotate seat to original transfer position.
- Do raise/lower seat to proper transfer height.
- Do unfasten seat belt or optional stability vest – transfer off seat.

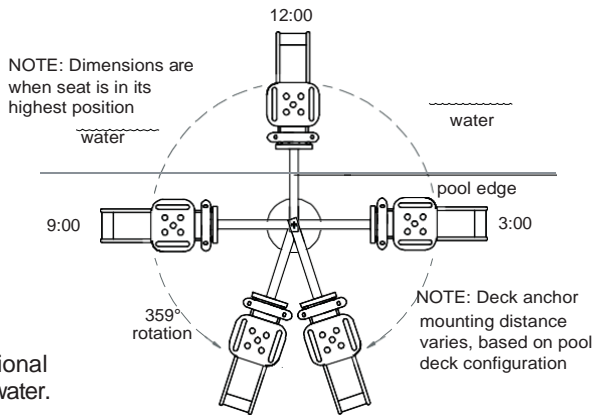

MAINTENANCE and CLEANSING

- Do keep all electronic components clean and dry.
- Do not allow disinfection solution to have direct contact with the skin and eyes.
- Do refer to Owner's manual for weekly / monthly hoist maintenance and cleansing.

Perform the following schedule to ensure proper operation before each use:

Maintenance Performed Before Each Use	Daily	Weekly	Monthly
Check battery level before each use / Charge battery daily	✓		
Wipe Control Box and battery connection with a clean dry rag	✓		
Examine hoist for any damage, loose or missing hardware	✓		
Test all control buttons (hand and control box) for normal operation	✓		
Spray gear assembly with a heavy-duty rust inhibitor/lubricant such as LPS 3 - Heavy-Duty Inhibitor		✓	
Make sure all cable connections are properly secured			✓
Inspect hoist frame, mast, support arm and seat assembly for rust			✓
Cleansing Performed After Each Use			
Rinse seat and seatbelt/stability vest with fresh water between each use - Cleanse seat and seatbelt/stability vest with a disinfectant solution of 1:100 dilution of household bleach to fresh water and then rinse with fresh water and dry entire hoist daily. In the event of a contamination incident such patient/user excreta - cleanse seat and seatbelt/stability vest immediately with the disinfectant solution*	✓		
Cleanse all battery connections with a nylon scouring pad			✓
Cleanse all metallic surfaces with a cleaner wax to maintain the finish of the hoist			✓

* When using the disinfection solution, avoid direct contact with the skin and eyes. In the event of a contamination incident, immerse the seat belt or stability vest in the disinfection solution for 10 min. and then rinse thoroughly with fresh water.